

User manual

2019-06-24

CuroCell® AREA Zone

Self-adjusting pressure care mattress

User manual item number: 95-001222



Contents

| | |
|------------------------------|----|
| Important safety precautions | 3 |
| Introduction | 4 |
| Product description | 7 |
| Instruction for use | 8 |
| Cleaning and reconditioning | 9 |
| Storage | 10 |
| Troubleshooting | 10 |
| Technical description | 10 |
| Spare parts | 12 |
| Options and accessories | 13 |
| Warranty | 14 |
| Other information | 14 |
| Own notes | 15 |

IMPORTANT SAFETY PRECAUTIONS

Read all instructions prior to use

WARNING!

- 1 The product must only be used for its intended purpose as described in this manual. The product may not be combined, assembled or repaired with other parts, accessories or spare parts other than those described in this manual or other documentation from Care of Sweden.
- 2 The mattress are protected with a hygiene cover; Please try to avoid the simultaneous use of multiple hygiene cover with this mattress. Because of the risk vapour permeable can be destroyed.
- 3 DO NOT USE in the vicinity of or in contact with fire sources/hot surfaces, such as fire, burning cigarettes or hot lamps. Although the product has undergone fire testing, it could become damaged if it comes into contact with fire sources.
- 4 Position and use the product in a manner that prevents it from becoming trapped or damaged. NOTE: Be particularly aware of trapping damage when using bed rails.
- 5 Always make sure the mattress has the right size.
- 6 The hygiene cover does not allow liquid or air to penetrate, but is vapour permeable. Make sure that the user is positioned correctly to avoid the risk of suffocation.
- 7 Prevent damage to the hygiene cover from sharp objects.
- 8 If the cover is supplied with handles on the long sides, these handles are only to be used for lifting and moving the mattress itself. All other use will be at your own responsibility and is not covered by the product warranty.
- 9 Follow the operations and maintenance instructions carefully in order to prevent bodily injuries, as well as damages to the product or other equipment.

1. Introduction



Always read the user manual prior to use.

1.1 General information

This CuroCell® mattress is a medical device in accordance to MDD 93/42/EEC. According to statutory regulations made by the authorities regarding medical devices, the manufacturer is required to report all accidents or incidents involving these products. We would be very grateful for all information involving accidents or incidents relating to our products, by reporting them immediately to us, here at Care of Sweden. The system has been tested and approved according to the following standards: SS-EN ISO 12182, SS-EN ISO 14971, SS-EN ISO 10993, EN 597-1, EN 597-2, EN 14126.

1.2 Intended use

CuroCell® AREA Zone is a replacement mattress that can be used in a range of health care environments to prevent pressure ulcers as well as treatment up to and including category 3⁽¹⁾. The recommended user weight is 0-230 kg.

We recommend that position changes are made regularly. The time interval must be evaluated by the responsible personnel, depending on the patient's status, diagnosis and general condition.

The mattress must always be cleaned between patients.

NOTE!

- When used together with positioning pillows, read the pillow user manual carefully for correct positioning of the patient.
- If you are using this product with an evacuation sheet, it is the user's responsibility to ensure that it is safe to use for evacuation.
- Be careful when using bed rails or other protection on the bed so that the mattress does not become crushed or damaged.
- The mattress must not be lifted with a person lying on it or used for any other type of transport other than a possible evacuation using an evacuation sheet.
- This mattress is unsuitable for use during x-ray examinations because of the risk of blurred images or artefacts that may lead to diagnostic errors.

(1) National Pressure Ulcer Advisory Panel, European Pressure Ulcer Advisory Panel and Pan Pacific Pressure Injury Alliance. Prevention and Treatment of Pressure Ulcers: Quick Reference Guide. Emily Haesler (Ed.). Cambridge Media: Osborne Park, Western Australia; 2014.

1.3 Evacuation instructions


Evac is a mattress cover with a built-in function for evacuation of bed-bound users. In case of fire or other emergency, there is no time to think about what to do next. Therefore, it is very important that the safety equipment, as well as the evacuation procedures, are well planned and known in advance.

The Evac cover is equipped with three handles on the head and foot end. On the sides, there are handles with Velcro straps. These straps are to be used to fixate the user to the mattress before commencing the evacuation. The straps are folded in the protective pockets on the side of the mattress with the ends sticking out to ensure that they are easy to pull out.

WARNING!


Person to perform an evacuation needs to have the necessary training. Always check the equipment, cover and mattress before commencing, to make sure an evacuation can be carried out safely. It is the user's responsibility to ensure that it is safe to use for evacuation.

Evacuation instructions

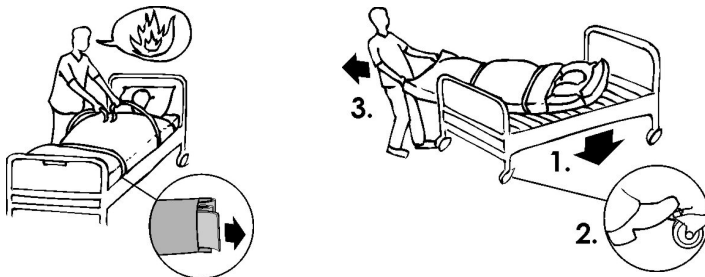

Down from the bed

- Inform the patient about what will happen.
- Lower and lock the bed in position.
- Disconnect the tubes from the pump.
- Pull out and fasten the Velcro straps.
- Use the outer handles at the foot end of the mattress and turn the mattress out from the bed.
- Slowly pull the patient down on to the floor.

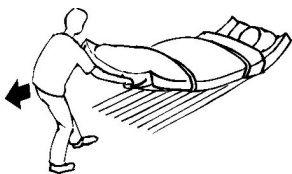

On an even surface

- Pull on the outer handles at the foot end of the mattress.
- Working position should be a little bit leaning back.

Down the stairs

- Walk backwards until the main part of the mattress has slide pass the top stair.
- Turn the back to the patient and look in the moving direction.
- Hold one hand on the stair handle and pull with the other hand on the middle handle of the mattress.
- Walk down the stairs. It is important to maintain speed, without running.



Training/practise

To bear in mind when training.

Pulling down from the bed:

- The patient in the bed should not be at the very top of the mattress. There could be a risk of hitting the head in the bed when pulling down.

Pulling in the stairs:

- Start with pulling an empty mattress down the stairs, in order to practice the technique.
- When training with a person on the mattress, a string or rope can be fasten in the handles at the head end. Another person can then walk behind the mattress and help out if problems should occur.
- It is advisable to practice in the same stairs where an evacuation can be necessary.

General:

- The main purpose of the training is not to reach high speed, but to learn the correct technique.

Training covers

- Because of the heavy strain on the equipment when evacuating, always check the seams and handles at the end of training. We recommend that equipment used for training is marked with “Training equipment” and is only used for this purpose.

Explanations

- If the Velcro straps are not used, there is a risk of the patient sliding/rolling down from the mattress.
- Pulling at the foot end reduces the strain on the puller, which means faster and safer evacuation is possible.
- Pulling in the stairs is the most difficult part of the evacuation and should be practiced extra carefully.
- The arms of the patient shall be inside of the Velcro straps. This way reduces the risk of the user grabbing hold to something out of fear.

NOTE!

When using wider mattresses, make sure that the evacuation routes are clear (size of doors etc.). Responsible personnel have to ensure that the evacuation can be performed safely. To ensure proper function of the Evac cover, it is important to regularly check the quality of the mattress. Check for damages that could cause problems when evacuating.

1.4 Other information

Action plan Pressure ulcers

To achieve the best possible result for this product it is recommended that a well-planned and well-structured working environment be used. In our guideline "Action plan" you will find examples which can be applied and put into practice when using our products.

Mattress guide

This practical guide gives you as a care giver, help in choosing the correct mattress from the Care of Swedens range. CuroCell® AREA Zone is a mattress in function group B. This information can be found on our website www.careofsweden.com or you can order a copy from our Customer services by sending your details to info@careofsweden.se.

2. Product description

Because of its unique construction combining foam and air, this mattress provides high user comfort, as well as a solution for pressure care without the need for external pump or electricity. The system uses patient weight and movements to distribute the internal air reservoir amongst the cells, in order to always maintain low surface pressure.

The CuroCell® AREA Zone is produced in high quality materials for function and durability. Foam part in two layers. Zone profiled top part.

All mattresses have a sloped heel section for protection of heels.

Available sizes are (width, length and height) 80/85/90/105/120 x 200/210 x 18 cm. For request of other sizes please contact Care of Sweden or your local dealer.

2.1 Description covers

CuroCell® AREA Zone is delivered with hygiene cover Steel/Duro. It is easy to handle and maintenance and get through highly set requirements of cleaning and hygiene. The cover is vapor permeable, which reduces risk of skin maceration. An inner cover protects foam and cells.

Cover Steel/Duro is a two-part cover with welded seams and a protected zipper. Has functional handles for transportation of the mattress. Colour: blue.

An inner cover is also included.

Cover Steel/Evac

Cover with welded seams and a protected zipper on all sides, divisible (top and bottom part). Functional handles for evacuation, moving and transport. Colour: blue/black.

Cover Stone/Duro


Crib 5 cover with welded seams and a protected zipper on all sides, divisible (top and bottom part). Colour grey/blue.

Cover Stone/Evac

Crib 5 cover with welded seams and a protected zipper on all sides, divisible (top and bottom part). Functional handles for evacuation, moving and transport. Colour grey/black.

2.2 Description mattress

CuroCell® AREA Zone

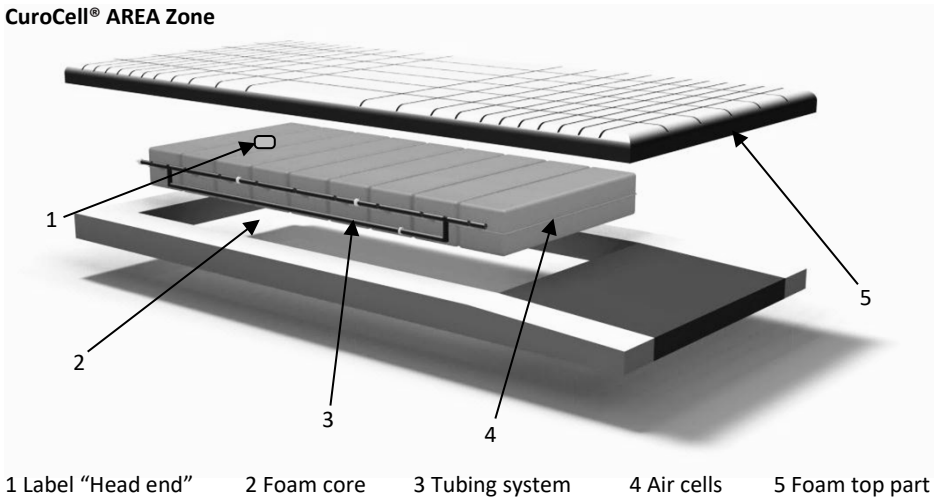

3. Instruction for use

3.1 Unpacking the mattress

The mattress is packed in a box.


1. Start with making sure that the package is not damaged. When opening the box and any other protective material, avoid using a knife since this could damage the mattress and cover.
2. Put the mattress on the bed frame and correct the cover if necessary.
3. After a few minutes the product is ready to be used.

3.2 Hand check

Hand check is used to ensure that the function of the mattress is correct. This should be done regularly.

Open the cover and insert your hand between the air cells directly underneath the user's pelvic region, use a vertical hand. There should be a clearance of 2-3 fingers between mattress base and the patient. If not, you need to check cells and tubing.

If the problem is not resolved, please contact Care of Sweden or your local dealer.

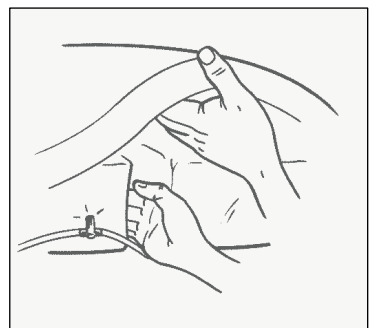

4. Cleaning and reconditioning

It is important to follow the instructions below before using the product for a different user. Consult the hygiene responsible or the supplier for help and instructions when unsure.

NOTE!

Check the hygiene cover each time the product is cleaned. If damaged, it must be replaced or repaired.

4.1 Covers

MATTRESS COVER



Covers can be wiped off with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70 %, oxidating solutions or chlorine max 1 %. Excrements and blood spots must be removed immediately with cold water. Carefully follow local instructions and usage instructions for the cleaner.



Multi-layer covers should be divided before washing. Wash with similar colours.



Chlorine and phenol-based cleaners could have a negative impact on the PU surface and should be avoided. If chlorine is used, we recommend a mixture of max. 1 %.

INNER COVER



Clean the affected area with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70 %, oxidating solutions or chlorine max 1 %.

4.2 Foam core



Clean the affected area with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70%, oxidating solutions or chlorine max 1%.

Press out the water carefully. Let the foam core dry in a warm, ventilated area, without direct exposure of sunlight. The foam core needs to be completely dry before it is used again.

4.3 Air cells

1. Remove the covers. See instructions in chapter 4.1 for cleaning covers.
2. Carefully remove the air cells from the foam core. Remove the cell holder from the cells.
3. Place the air cells and the tubes on a clean area which is suitable for washing.
4. Wipe off the cells, all tubes with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70 %, oxidating solutions or chlorine max 1 %.
5. Replace the cells into the cell holder. Place the air cells back into the foam core. Check that all the tubes and connectors are secure and tight, before closing the cover.
6. Wipe off work surfaces with alcohol-based disinfectant/cleaner intended for this purpose.

5. Storage

Store the mattress on a flat surface or carefully rolled/folded with the zipper closed. Always store at room temperature in a dry area. Make sure the mattress is stored where it will not be exposed to thrusts, hits or other things that can damage it. Avoid direct exposure of sunlight. If the mattress is to be stored more than 3 months, we recommend it is stored on a flat surface (not folded or rolled up).

6. Troubleshooting

| Problem | Solution |
|---------|----------|
|---------|----------|

| | |
|------------------------------|--|
| The patient is bottoming out | Open covers and check cells and tubing. Check again through Hand check (see chapter 3.2). |
|------------------------------|--|

If your questions are not answered by information above, please contact Care of Sweden or your local dealer.

7. Technical description

7.1 Technical specification

| MATTRESS | SPECIFICATION |
|-------------------|------------------------------------|
| Model | CuroCell® AREA Zone |
| Dimension (WxLxH) | 80/85/90/105/120 x 200/210 x 18 cm |
| Weight | 16 kg (90x200x18 cm) |

NOTE: The manufacturer retains the right to modify the product specification at any time.

7.2 Labelling

The mattress is labelled with product name, production date, article number, size, care and wash instructions, EAN-bar code, as well as space for own labelling. The product also displays which side is up/down and head/foot.

7.3 Symbol Definitions



Article No



Manufacturer

XXXX-XX-XX

Year-Month-Day



Foot placement



Read the user manual



Group of function



CE marked acc. to
MDD 93/42/EEC



Recycling



Not in domestic refuse,
follow recycling instructions



Recommended user weight



Patient information
– category



Directly on bed base



Recommended user
weight when using Evac
function



Counteract shear force



Heel function



Do not wash



Machine wash at 95 °C



<1%
Chlorine



Tumble dry



Do not tumble dry



Drip dry



Do not iron



Do not dry clean



Wipe off



Class II equipment
(Double Insulated)




Type BF



IP-class
(Ingress Protection Rating)



Temperature range

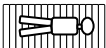

The mattress should only be
used lying in the lengthwise
direction on the mattress

8. Spare parts

Cover Steel/Duro and Stone/Duro for CuroCell AREA Zone

| Size | Top part Steel Art.no. | Top part Stone Art.no. | Bottom part Duro Art.no. |
|-------------|-----------------------------------|-----------------------------------|-------------------------------------|
| 80x200 cm | CC-925511 | CC-925581 | CC-925011 |
| 80x210 cm | CC-925512 | CC-925582 | CC-925012 |
| 85x200 cm | CC-925513 | CC-925583 | CC-925013 |
| 85x210 cm | CC-925514 | CC925584 | CC-925014 |
| 90x200 cm | CC-925515 | CC-925585 | CC-925015 |
| 90x210 cm | CC-925516 | CC-925586 | CC-925016 |
| 105x200 cm | CC-925517 | CC-925587 | CC-925018 |
| 120x200 cm | CC-925519 | CC-925589 | CC-925019 |

| Art. no. | Description | Size |
|-----------------|----------------------|-------------|
| CC-900050 | Air cell AREA, 1 pcs | 63x15x12 cm |

If other spare parts are needed, please contact Care of Sweden or your local dealer for further assistance.

9. Options and accessories

9.1 Options

Sturdy transport and storage bag

| Art. No. | Description | Width |
|-----------|-------------------------|------------|
| 58-900905 | Mattress bag | 105-120 cm |
| 58-900906 | Mattress bag extra high | 80-90 cm |
| 58-900911 | Mattress bag extra high | 105-120 cm |

9.2 Accessories

Cover Steel/Evac and Stone/Evac for CuroCell AREA Zone

| | Top part Steel | Top part Stone | Bottom part Evac | Complete cover Steel/Evac | Complete cover Stone/Evac |
|------------|----------------|----------------|------------------|---------------------------|---------------------------|
| Size | Art.no. | Art.no. | Art.no. | Art.no. | Art.no. |
| 80x200 cm | CC-926110 | - | 461-080200 | CC-925940 | - |
| 80x210 cm | CC-926111 | - | 461-080210 | CC-925941 | - |
| 85x200 cm | CC-926112 | CC-925586 | 461-085200 | CC-925942 | CC-925862 |
| 85x210 cm | CC-926113 | - | 461-085210 | CC-925943 | - |
| 90x200 cm | CC-926114 | CC-925587 | 461-090200 | CC-925944 | CC-925864 |
| 90x210 cm | CC-926115 | - | 461-090210 | CC-925945 | - |
| 105x200 cm | CC-926116 | CC-925588 | 461-105200 | CC-925946 | CC-925866 |
| 105x210 cm | CC-926117 | - | 461-105210 | CC-925947 | - |
| 120x200 cm | CC-926118 | CC-925589 | 461-120200 | CC-925948 | CC-925868 |
| 120x210 cm | CC-926119 | - | 461-120210 | CC-925949 | - |

Inner cover elastic for CuroCell AREA Zone

| Art. No. | Size | Art. No. | Size |
|-----------|-----------|-----------|------------|
| CC-928250 | 80x200 cm | CC-928255 | 90x210 cm |
| CC-928251 | 80x210 cm | CC-928256 | 105x200 cm |
| CC-928252 | 85x200 cm | CC-928257 | 105x210 cm |
| CC-928253 | 85x210 cm | CC-928258 | 120x200 cm |
| CC-928254 | 90x200 cm | CC-928259 | 120x210 cm |

9.3 Other accessories

To ensure the best level of comfort, we recommend use of accessories from Care of Sweden since our products are designed to work well together. Please contact Care of Sweden or your local dealer for accessories or more information on our products.

10. Warranty

10.1 Scope

This CuroCell® system is covered by a 5-year warranty for manufacturing defects. The warranty does not apply to normal wear and tear or damage due to negligence or improper handling/care.

10.2 Service and repairs

Service and maintenance must always be performed by Care of Sweden or one of its authorised technicians. Only use spare parts that have been approved by Care of Sweden. Use of non-approved spare parts invalidates the warranty.

A warranty claim must be submitted to Care of Sweden before the product is returned.

Please contact Care of Sweden or your local dealer if you have questions regarding warranty and service.

10.3 Life expectancy

This product's expected life is estimated to be 5 years.

11. Other information

11.1 Disassembling and recycling

A used CuroCell® mattress must be taken to a recycling centre and sorted as combustible waste.

For more information contact Care of Sweden or your local dealer.



Manufactured by

 care of sweden

SUPPORTING LIFE

Contact:

Phone: + 46 771 106 600

Fax: + 46 325 12840

Email: info@careofsweden.se

Internet: www.careofsweden.com

Address:

Care of Sweden AB
P.O. Box 146
SE-514 23 Tranemo
SWEDEN

Visit address:

Fabriksgatan 5A
SE-514 33 Tranemo
SWEDEN

Cargo address:

Byns väg 4A
SE-514 33 Tranemo
SWEDEN

Supplied by