

Curera[®] Support Wedge

For positioning & position changing



 **curera**[™]
positionering

 care of sweden
SUPPORTING LIFE

Curera® Support Wedge

If a person makes too few spontaneous movements, there is an increased risk of pressure ulcers. Regular changes of position reduce the risk of pressure ulcers. Position as required in both reclining and seated position.

Curera® Positioning is a range of stable pillows, rolls, and wedges that meet most needs for support and relief. They are also a great help when changing dressings and for patient care.

Curera® Support Wedge can be used for placing the patient/user in a lateral position to relieve the sacrum, which is a particularly sensitive area. Two support wedges must be used together for lateral positioning.

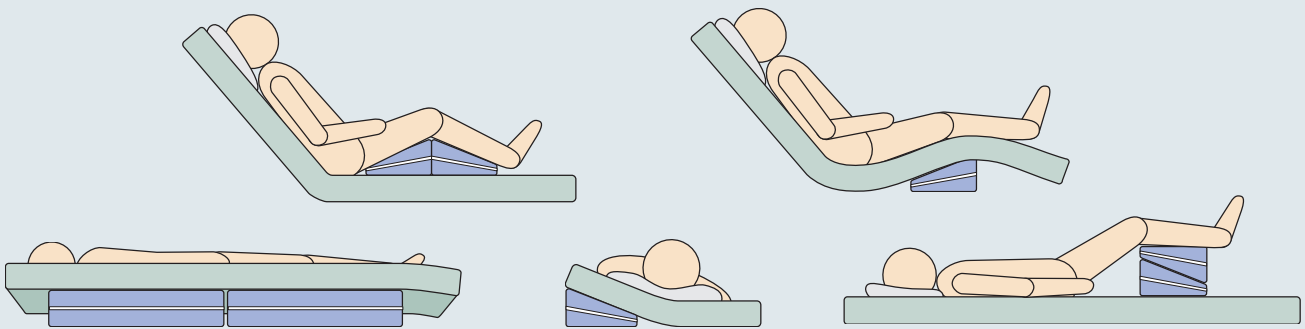
The Support Wedge comes with a Hygiene Cover. The hygiene cover does not allow liquid or air to penetrate, but is vapour permeable (vapour is wicked away).



Make sure that the patient is positioned correctly so that airways are unobstructed.

The support wedges are to be placed differently, depending on the mattress being used. Refer to the user instructions for more information.

Working with Curera® Positioning is simple, and the benefits are many. Contact Care of Sweden for training and further information.

POSITIONING EXAMPLE



CLEANING	FACTS
<p>The Hygiene Cover can be wiped clean with detergent and/or disinfectant. Machine wash at max. 95 °C.</p>	<ul style="list-style-type: none"> • Filling: 100 % cold foam • Hygiene cover: Polyurethane/polyester • Warranty: 1 year • Sizes: 50x26x15/5 cm & 70x26x15/5 cm <div style="display: flex; align-items: center; margin-top: 20px;"> <div style="margin-right: 10px;">  </div> <div style="display: flex; align-items: center;">  <div style="margin-left: 5px;"> <p>Read the user manual prior to using the product.</p> </div> </div> </div>
EXAMPLES OF USE	
<ul style="list-style-type: none"> • For support and relief in the prevention of pressure ulcers • When it is difficult for a user to lie fully on their side • When a user has difficulty changing position themselves • To protect the user from kicking against the bed rail 	

Care of Sweden and Curera are registered trademarks of Care of Sweden AB. © Care of Sweden AB, 2019. All rights reserved. Reproduction prohibited. Last updated: 2019-07-03