


Instructions for use

2021-07-07

Curera® Seat cushions

Applies to the following products:

- Curera® Consil
- Curera® Contur
- Curera® Contur EX
- Curera® Plantur
- Curera® Consept
- Curera® Consept EX
- Curera® Consept Latus
- Curera® Hemi
- Curera® Tremix
- Curera® Contur Children
- Curera® Seat cushion
- CuroCell® S.A.C.
- Standard Cover
- Deluxe Cover
- Hood Cover
- Hygienic Cover
- Onyx Cover
- Onyx+ Cover
- Levelling plate

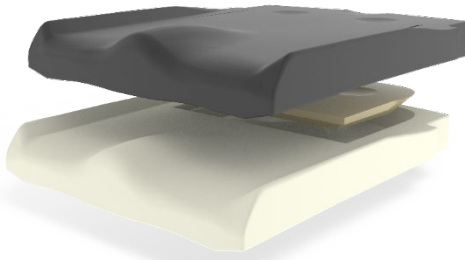

Instructions for use item number: 97-001003



Contents

Warning and safety precautions	3
Introduction	3
Product description	4
Use	12
Reuse, cleaning and reconditioning	14
Other information	15
Accessories	17
Own notes	19

WARNINGS- AND SAFETY PRECAUTIONS

Read all instructions prior to use

WARNING!

1. The product must only be used for its intended purpose as described in this user guide. The product may not be combined, fitted or repaired with other parts, accessories or spare parts other than those described in this user guide or other documentation published by Care of Sweden.
2. If the cushion is protected by a hygiene cover, avoid combining it with other fully covering plastic protectors. The cover's vapour permeability may be affected.
3. DO NOT USE in the vicinity of or in contact with naked flames/hot surfaces, such as fire, burning cigarettes or hot lamps. Although the product has undergone fire testing, it could become damaged if it comes into contact with naked flames.
4. The product must be placed and used so that it does not become trapped or damaged.
5. If the cushion has been exposed to high temperatures, e.g. if it has been stored in direct sunlight, you must check that it is not too hot to sit on prior to use.
6. Always check that the cushion is of the correct size.
7. Before use, check the cushion to ensure that it is not damaged.
8. Avoid damaging the hygiene cover using sharp objects.
9. Extended periods of sitting increase the risk of DVT (deep-vein thrombosis).
10. Regularly check the skin for redness. If redness occurs, stop using the cushion immediately and contact your prescriber.
11. The cushion is designed to be used in a wheelchair or a chair with back and arm supports. When using in a wheelchair with a suspension kit, a balancing plate is available as an accessory.
12. Cushions must always be used with a cover.
13. Check to ensure that the cover fits the cushion well - it must always be a good fit.

1 Introduction



Always read the instructions for use prior to use.

1.1 General information

The Curera® seat cushions are medical devices and are CE-marked in accordance to MDR (EU) 2017/745. According to statutory regulations made by the authorities regarding medical devices, the manufacturer is required to report all accidents or incidents involving these products. We would be very grateful for all information involving accidents or incidents relating to our products, by reporting them immediately to us, here at Care of Sweden.

1.2 Intended use

Curera® seat cushions are intended to be used for positioning, as an aid for prevention of pressure ulcers/pressure injuries (PU/PI)

1.3 Intended user

Curera® seat cushions are intended to be used by all kind of users, including lay persons.

1.4 Intended use environment

Curera® seat cushions are designed to be used in wheelchairs and on furniture, for pressure redistribution and positioning, in all kinds of healthcare environments, including home care.

1.5 Other information

To achieve the best possible result for this product it is recommended that a well-planned and well-structured working environment be used. In our guideline "Action plan" you will find examples which can be applied and put into practice when using our products.

2 Product description

Curera seat cushions are available in a large numbers of different designs and sizes. The cushion can be used in all segments of the healthcare market and are especially suitable for use with wheelchairs. For details of the standard sizes available. See paragraph 2.2. For other sizes, please contact Care of Sweden or your local dealer.

2.1 Covers

All covers are easy to manage and handle and conform to stringent requirements in respect of cleaning and hygiene. For more information about cleaning, see paragraph 4.

Covers for Curera® Consil, Curera® Contur, Curera® Contur EX, Curera® Plantur, Curera® Concept, Curera® Concept EX, Curera® Concept Latus, Curera® Hemi, Curera® Tremix, Curera® Contur Children:

Standard Cover - A detachable plush polyester cover (100% polyester) with a zipper on the rear edge.

Hygienic cover – Cover in polyurethane-coated polyester. A liquid proof cover that is easy to maintain.

Hood Cover - Made from 2-way stretch polyester.

Deluxe Cover - A detachable cover with plush polyester outside and a self-ventilating material inside. Zipper on rear edge.

Onyx cover – Flexible, elastic and detachable cover made of a 4-way stretch on the top side that prevents the user sliding and an underside made from an anti-slip material. Handle on the underside.

Onyx+ cover – Flexible, elastic and detachable cover made of a 4-way stretch on the top side that prevents the user sliding and an underside made from an anti-slip material. Handle on the

underside. A layer in between, that is made of mesh material for a better microclimate and a higher comfort.

Covers for Curera® Seat Cushion and CuroCell® S.A.C.:

Hygienic cover – Cover in polyurethane-coated polyester. Liquid proof.

2.2 Product descriptions

Curera® Consil

A pressure redistributing and anatomically shaped seat cushion.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Consil, including Hood cover:

Item no.	Size (WxDxH)	Rec. Max user weight/Kg	Item no.	Size (WxDxH)	Rec. Max user weight/Kg
803535-40	35x35x5 cm	60	804245-40	42x45x5 cm	125
803540-40	35x40x5 cm	60	804250-40	42x50x5 cm	125
803740-40	37x40x5 cm	90	804540-40	45x40x5 cm	125
803745-40	37x45x5 cm	90	804545-40	45x45x5 cm	125
804040-40	40x40x5 cm	90	804550-40	45x50x5 cm	125
804045-40	40x45x5 cm	90	804740-40	47x40x5 cm	125
804050-40	40x50x5 cm	90	804745-40	47x45x5 cm	150
804240-40	42x40x5 cm	125	804750-40	47x50x5 cm	150
804245-40	42x45x5 cm	125	805040-40	50x40x5 cm	150
804250-40	42x50x5 cm	125	805045-40	50x45x5 cm	150
804540-40	45x40x5 cm	125	805050-40	50x50x5 cm	150

Item information Consil, including Standard cover:

Item no.	Size (WxDxH)	Rec. Max user weight/Kg	Item no.	Size (WxDxH)	Rec. Max user weight/Kg
513035-11	30x35x5 cm	70	514740-11	47x40x5 cm	125
513535-11	35x35x5 cm	70	514745-11	47x45x5 cm	150
513540-11	35x40x5 cm	70	514750-11	47x50x5 cm	150
513545-11	35x45x5 cm	70	515040-11	50x40x5 cm	150
513740-11	37x40x5 cm	100	515045-11	50x45x5 cm	150
513745-11	37x45x5 cm	100	515050-11	50x50x5 cm	150
513750-11	37x50x5 cm	100	515240-11	52x40x5 cm	150
514040-11	40x40x5 cm	110	515245-11	52x45x5 cm	150
514045-11	40x45x5 cm	110	515250-11	52x50x5 cm	150
514050-11	40x50x5 cm	110	515540-11	55x40x5 cm	150
514240-11	42x40x5 cm	125	515545-11	55x45x5 cm	150
514242-11	42x42x5 cm	125	515550-11	55x50x5 cm	150
514245-11	42x45x5 cm	125	515555-11	55x55x5 cm	150
514250-11	42x50x5 cm	125	515740-11	57x40x5 cm	150
514343-11	43x43x5 cm	125	515745-11	57x45x5 cm	150
514540-11	45x40x5 cm	125	515750-11	57x50x5 cm	150
514545-11	45x45x5 cm	125	516045-11	60x45x5 cm	150
514547-11	45x47x5 cm	125	516050-11	60x50x5 cm	150
514550-11	45x50x5 cm	125	516545-11	65x45x5 cm	150

Curera® Contur

A pressure redistributing seat cushion, constructed from an anatomically moulded polyurethane foam base with viscose elastic foam on the underside.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 5 cm

Warranty: 2 years

Fire: SS-EN 1021-1, SS-EN 1021-2, ISO 7176-16

Item information Contur, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
513035-11	30x35x5 cm	70	514740-11	47x40x5 cm	125
513535-11	35x35x5 cm	70	514745-11	47x45x5 cm	150
513540-11	35x40x5 cm	70	514750-11	47x50x5 cm	150
513545-11	35x45x5 cm	70	515040-11	50x40x5 cm	150
513740-11	37x40x5 cm	100	515045-11	50x45x5 cm	150
513745-11	37x45x5 cm	100	515050-11	50x50x5 cm	150
513750-11	37x50x5 cm	100	515240-11	52x40x5 cm	150
514040-11	40x40x5 cm	125	515245-11	52x45x5 cm	150
514045-11	40x45x5 cm	125	515250-11	52x50x5 cm	150
514050-11	40x50x5 cm	125	515540-11	55x40x5 cm	150
514240-11	42x40x5 cm	125	515545-11	55x45x5 cm	150
514242-11	42x42x5 cm	125	515550-11	55x50x5 cm	150
514245-11	42x45x5 cm	125	515555-11	55x55x5 cm	150
514250-11	42x50x5 cm	125	515740-11	57x40x5 cm	150
514343-11	43x43x5 cm	125	515745-11	57x45x5 cm	150
514540-11	45x40x5 cm	125	515750-11	57x50x5 cm	150
514545-11	45x45x5 cm	125	516045-11	60x45x5 cm	150
514547-11	45x47x5 cm	125	516050-11	60x50x5 cm	150
514550-11	45x50x5 cm	125	516545-11	65x45x5 cm	150

Curera® Contur EX

A pressure redistributing cushion, constructed from an anatomically moulded polyurethane foam base with viscose elastic foam on the underside.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, elastic foam

Height: 5 cm

Warranty: 2 years

Fire: SS-EN 1021-1, SS-EN 1021-2, ISO 7176-16

Item information Contur EX, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
513535-25	35x35x5 cm	70	514540-25	45x40x5 cm	125
513540-25	35x40x5 cm	70	514545-25	45x45x5 cm	125
513740-25	37x40x5 cm	100	514550-25	45x50x5 cm	125
513745-25	37x45x5 cm	100	514740-25	47x40x5 cm	125
514040-25	40x40x5 cm	125	514745-25	47x45x5 cm	150
514045-25	40x45x5 cm	125	514750-25	47x50x5 cm	150
514050-25	40x50x5 cm	125	515040-25	50x40x5 cm	150
514240-25	42x40x5 cm	125	515045-25	50x45x5 cm	150
514245-25	42x45x5 cm	125	515050-25	50x50x5 cm	150
514250-25	42x50x5 cm	125			

Curera® Plantur

A pressure redistributing seat cushion, constructed from anatomically moulded polyurethane foam with a flat top side and a viscose elastic foam insert in the underside.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Plantur, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
523035-20	30x35x5 cm	70	524250-20	42x50x5 cm	125
523535-20	35x35x5 cm	70	524540-20	45x40x5 cm	125
523540-20	35x40x5 cm	70	524545-20	45x45x5 cm	125
523545-20	35x45x5 cm	70	524550-20	45x50x5 cm	125
523740-20	37x40x5 cm	100	524740-20	47x40x5 cm	150
523745-20	37x45x5 cm	100	524745-20	47x45x5 cm	150
523750-20	37x50x5 cm	125	525040-20	50x40x5 cm	150
524040-20	40x40x5 cm	125	525045-20	50x45x5 cm	150
524045-20	40x45x5 cm	125	525050-20	50x50x5 cm	150
524050-20	40x50x5 cm	125	525540-20	55x40x5 cm	150
524750-20	47x50x5 cm	150	525545-20	55x45x5 cm	150
524240-20	42x40x5 cm	125	525550-20	55x50x5 cm	150
524245-20	42x45x5 cm	125			

Curera® Concept

A pressure redistributing seat cushion, constructed from an anatomically moulded polyurethane foam base with viscose elastic foam insert on the top side.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Concept, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
543535-11	35x35x5 cm	70	544545-11	45x45x5 cm	130
543540-11	35x40x5 cm	70	544550-11	45x50x5 cm	130
543545-11	35x45x5 cm	70	544740-11	47x40x5 cm	130
543740-11	37x40x5 cm	105	544745-11	47x45x5 cm	150
543745-11	37x45x5 cm	105	544750-11	47x50x5 cm	150
544040-11	40x40x5 cm	115	545040-11	50x40x5 cm	150
544045-11	40x45x5 cm	115	545045-11	50x45x5 cm	150
544050-11	40x50x5 cm	115	545050-11	50x50x5 cm	150
544240-11	42x40x5 cm	130	545545-11	55x45x5 cm	150
544245-11	42x45x5 cm	130	545550-11	55x50x5 cm	150
544250-11	42x50x5 cm	130	545745-11	57x45x5 cm	150
544540-11	45x40x5 cm	130			

Curera® Consept EX

A pressure redistributing cushion, constructed from an anatomically moulded polyurethane foam base with viscose elastic foam the topside.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, elastic foam

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Consept EX, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
543535-20	35x35x5 cm	70	544540-20	45x40x5 cm	130
543540-20	35x40x5 cm	70	544545-20	45x45x5 cm	130
543740-20	37x40x5 cm	105	544550-20	45x50x5 cm	130
543745-20	37x45x5 cm	105	544740-20	47x40x5 cm	130
544045-20	40x45x5 cm	115	544745-20	47x45x5 cm	150
544050-20	40x50x5 cm	115	545040-20	50x40x5 cm	150
544240-20	42x40x5 cm	130	545045-20	50x45x5 cm	150
544250-20	42x50x5 cm	130	545050-20	50x50x5 cm	150

Curera® Consept Latus

A pressure redistributing seat cushion designed for heavier users. It is constructed from an anatomically moulded polyurethane foam base with a viscose elastic foam insert from the top side.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 7 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Consept Latus, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
585045-11	50x45x7 cm	250	585750-11	57x50x7 cm	260
585050-11	50x50x7 cm	250	585755-11	57x55x7 cm	260
585245-11	52x45x7 cm	250	586050-11	60x50x7 cm	260
585250-11	52x50x7 cm	250	586055-11	60x55x7 cm	260
585545-11	55x45x7 cm	250	586550-11	65x50x7 cm	270
585550-11	55x50x7 cm	250	586555-11	65x55x7 cm	270

Curera® Hemi

Hemi is a specially-designed seat cushion with a protected design (Reg.no. 79436) and suitable for people who need to be able to propel themselves in their wheelchair using their legs. Three versions are available - left, right and double.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Hemi left, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
513535-55	35x35x5 cm	70	514545-55	45x45x5 cm	130
513540-55	35x40x5 cm	70	514550-55	45x50x5 cm	130
514040-55	40x40x5 cm	115	515040-55	50x40x5 cm	150
514045-55	40x45x5 cm	115	515045-55	50x45x5 cm	150
514240-55	42x40x5 cm	130	515050-55	50x50x5 cm	150
514245-55	42x45x5 cm	130	516050-55	60x50x5 cm	150
514540-55	45x40x5 cm	130			

Item information Hemi right, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
513535-50	35x35x5 cm	70	514545-50	45x45x5 cm	130
513540-50	35x40x5 cm	70	514550-50	45x50x5 cm	130
514040-50	40x40x5 cm	115	515040-50	50x40x5 cm	150
514045-50	40x45x5 cm	115	515045-50	50x45x5 cm	150
514240-50	42x40x5 cm	130	515050-50	50x50x5 cm	150
514245-50	42x45x5 cm	130	516050-50	60x50x5 cm	150
514540-50	45x40x5 cm	130			

Item information Hemi double, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
513535-60	35x35x5 cm	70	514545-60	45x45x5 cm	130
513540-60	35x40x5 cm	70	514550-60	45x50x5 cm	130
514040-60	40x40x5 cm	115	515040-60	50x40x5 cm	150
514045-60	40x45x5 cm	115	515045-60	50x45x5 cm	150
514240-60	42x40x5 cm	130	515050-60	50x50x5 cm	150
514245-60	42x45x5 cm	130	516050-60	60x50x5 cm	150
514540-60	45x40x5 cm	130			

Curera® Tremix

An anatomically-designed seat cushion constructed in multiple layers and used to prevent pressure ulcers.

Classification: ISO code 04 33 03 (pressure relief)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 7 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Tremix, including Delux cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
563535-11	35x35x7 cm	70	564740-11	47x40x7 cm	130
563540-11	35x40x7 cm	70	564745-11	47x45x7 cm	150
563740-11	37x40x7 cm	105	564750-11	47x50x7 cm	150
563745-11	37x45x7 cm	105	565040-11	50x40x7 cm	150
564040-11	40x40x7 cm	115	565045-11	50x45x7 cm	150
564045-11	40x45x7 cm	115	565050-11	50x50x7 cm	150
564050-11	40x50x7 cm	115	564240-11	52x40x7 cm	150
564240-11	42x40x7 cm	130	565245-11	52x45x7 cm	150
564245-11	42x45x7 cm	130	565250-11	52x50x7 cm	150
564250-11	42x50x7 cm	130	565540-11	55x40x7 cm	150
564540-11	45x40x7 cm	130	565545-11	55x45x7 cm	150
564545-11	45x45x7 cm	130	565550-11	55x50x7 cm	150
564550-11	45x50x7 cm	130			

Curera® Contur Children

A pressure redistributing seat cushion, constructed from an anatomically moulded polyurethane foam base with a viscose elastic foam insert on the top side.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 4 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Contur Child, including Standard cover:

Item no.	Size (WxDxH)	Rec. max user weight/Kg	Item no.	Size (WxDxH)	Rec. max user weight/Kg
512525-12	25x25x4 cm	40	513035-12	30x35x4 cm	60
512530-12	25x30x4 cm	40	513335-12	33x35x4 cm	60
512725-12	27x25x4 cm	60	513535-12	35x35x4 cm	60
512730-12	27x30x4 cm	60	513540-12	35x40x4 cm	60
513030-12	30x30x4 cm	60	514040-12	40x40x4 cm	60
513034-12	30x34x4 cm	60			

Curera® Seat Cushion

A pressure redistributing seat cushion constructed with markings on the cushions which makes it easy to position.

Classification: ISO code 18 10 06 – pressure redistribution

Material in cushion: 100% flame-proof polyether foam

Height: 6 cm

Warranty: 1 year

Fire: ISO 7176-16

Item information Curera® Seat Cushion, including hygiene cover.

Item no.	Size (WxDxH)	Rec. max user weight/kg
52-035035	35x35x6 cm	120
52-035041	35x41x6 cm	120
52-040041	40x41x6 cm	120
52-040046	40x46x6 cm	120
52-040050	40x50x6 cm	120
52-045041	45x41x6 cm	120
52-045046	45x46x6 cm	120
52-045050	45x50x6 cm	120
52-050046	50x46x6 cm	120
52-050050	50x50x6 cm	120

CuroCell® S.A.C.

A static, redistributing seat cushion of air.

Material: Polyurethane

Classification: ISO code 04 33 03 (pressure relieving)

Warranty: 1 year

Fire: EN 597-1, EN 597-2

Item no.	Size	Rec. user weight/kg
CC-710-045045	45x45x5 cm	160 kg
CC-710-050050	50x50x5 cm	160 kg

Levelling Plate (Accessories)

There are two types of levelling plate, one placed inside the cover under the cushion, with the rounded rear edge facing backwards and the bevelled underside facing down against the underside of the cover. The other is placed under the wheelchair cushion to fill the gap between the cushion and sling seat, the underside is made from an anti-slip material. It can be used with all cushions except for the Tremix, which already have this feature.

Material in cushion: Cellpolyeten

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Levelling plate (without anti-slip):

Item no.	Size (WxDxH)	Item no.	Size (WxDxH)	Item no.	Size (WxDxH)
583535-30	35x35x2,5 cm	584540-30	45x40x2,5 cm	584750-30	47x50x2,5 cm
584040-30	40x40x2,5 cm	584545-30	45x45x2,5 cm	585045-30	50x45x2,5 cm
584045-30	40x45x2,5 cm	584550-30	45x50x2,5 cm	585050-30	50x50x2,5 cm
584240-30	42x40x2,5 cm	584740-30	47x40x2,5 cm	585550-30	55x50x2,5 cm
584245-30	42x45x2,5 cm	584745-30	47x45x2,5 cm	585555-30	55x55x2,5 cm

Item Information Levelling Plate (with anti-slip).

Item no.	Size (WxDxH)	Item no.	Size (WxDxH)
583040-20	30x40x2,5 cm	584545-20	45x45x2,5 cm
583540-20	35x40x2,5 cm	584550-20	45x50x2,5 cm
584040-20	40x40x2,5 cm	584750-20	47x50x2,5 cm
584045-20	40x45x2,5 cm	585045-20	50x45x2,5 cm
584540-20	45x40x2,5 cm	585050-20	50x50x2,5 cm
		585550-20	55x50x2,5 cm

3 Use

When unpacking, check to ensure that the item is not damaged. Remove the plastic bag and any protective material. Avoid using a knife as this risks damaging the product.


When using the products Curera® Consil, Curera® Contur, Curera® Contur EX, Curera® Plantur, Curera® Concept, Curera® Concept EX, Curera® Concept Latus, Curera® Hemi, Curera® Tremix, Curera® Contur Children:

1. Check that the cushion fits correctly in the cover. Follow the markings on the cover and cushion.
2. Place the cushion as indicated by the markings on the cushion in the wheelchair or armchair. Check that the cushion is the correct size for the chair seat.
3. The user can now start using the cushion.

Take Note! If the cushion Tremix has been stored in a temperature below + 10 °C, it shall be stored at room temperature for at least 1 h before being put into service.


Levelling Plate (without anti-slip)

Place the plate inside the cover, under the cushion, with the rounded rear edge facing backwards and the bevelled underside facing down against the underside of the cover. The plate is also labelled as to how it should be placed.

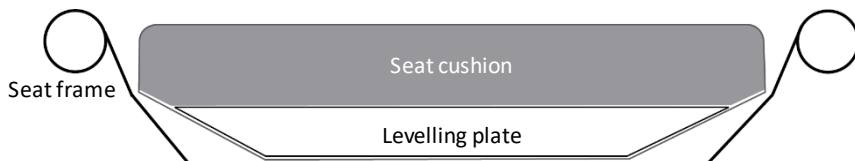

Levelling Plate (with anti-slip)

Place the plate under the wheelchair cushion with the anti-slip material facing downwards.

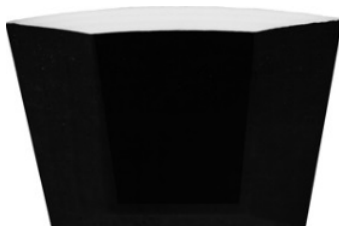

When using the product Curera® Seat Cushion:


1. Check that the cushion fits correctly in the cover. Follow the markings on the cover and cushion.
2. Place the cushion in the wheelchair with the “squares” facing upwards (feel with your hand) and the bevelled edge facing forwards.

Top





Front

3. The cushion is primarily intended for use on a flat seat. If it is a suspended seat with a drop (h) greater than about 25 mm, we recommend using a levelling plate.

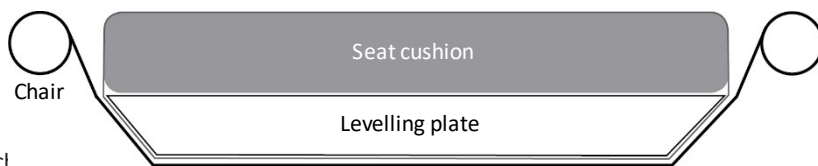

Attach the cushion to the cover by stretching the pressure points to the holes on the cover.

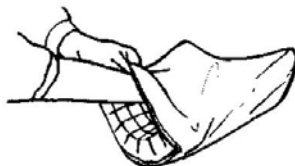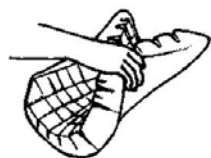

When using the product CuroCell® S.A.C.:

1. To check that the cushion is properly inflated roll the cushion from one side, it should be possible to roll the cushion to the second row of holes.
2. Place the cushion on the intended seat.
3. Perform a hand check to make sure that the cushion works properly. Do this by placing your hand between the base and the cushion to check the user does not sit through the cushion.

When using in a wheelchair with a suspension kit, a levelling plate is available as an accessory.

4 Reuse, cleaning and reconditioning

The products are reusable. When reusing, it is important to follow the instructions below for cleaning and reconditioning. Before using the product again for a new user, it is recommended to clean the cushion in accordance with the instructions below.

NOTE!

Carefully follow local instructions and cleaner usage instructions.

Standard Cover, Deluxe Cover and Hood Cover



The cover must be completely dry before being refitted. Pull the zipper (if the cover has one).

Hygienic Cover



Covers can be wiped off with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70 %, oxidating solutions or chlorine max 1 %. Excrements and blood spots must be removed immediately with cold water. Carefully follow local instructions and usage instructions for the cleaner.



Chlorine and phenol-based cleaners could have a negative impact on the PU surface and should be avoided. If chlorine is used, we recommend a mixture of max. 1%.

CUSHIONS - Curera® Contur, Curera® Contur EX, Curera® Plantur, Curera® Concept, Curera® Concept EX, Curera® Concept Latus, Curera® Hemi, Curera® Tremix, Curera® Contur Children, CuroCell® S.A.C.:



Clean the affect area with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70%, oxidating solutions or chlorine max 1 %.

Let the cushion dry in a warm, ventilated area, without direct exposure of sunlight. The cushion needs to be completely dry before it is used again.

CUSHIONS - Curera® Consil and Curera® Seat Cushion



Clean the affect area with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70 %, oxidating solutions or chlorine max 1 %.

Press out the water carefully. Don't wring or roll the cushion to extract water. Let the foam core dry in a warm, ventilated area, without direct exposure of sunlight. The foam core needs to be completely dry before it is used again.

Levelling plate (Accessory)



Clean the affect area with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70%, oxidating solutions or chlorine max 1 %.

It needs to be completely dry before it is used again.

5 Other information

5.1 Storage

The cushions should be stored in a dry, room temperature area. Store the cushion so that it cannot be exposed to shocks, impacts or other mechanical influences. Avoid exposing the cushion to direct sunlight.

5.2 Labelling

The cushion is labelled with date of manufacture, order number, item number, size, care and maintenance advice, EAN number. It is also marked showing how it must be positioned in the chair.

5.3 Warranty

The warranty is specified under chapter 2.2.

The warranty does not apply to normal wear and tear or product impairment or damage stemming from negligence or improper handling/care.

5.4 Service and maintenance

The cushions require no service or maintenance other than cleaning as described in this instructions for use.

5.5 Life expectancy

The expected lifetime for the cushions are estimated to be 2 years, except from CuroCell® S.A.C. The estimated lifetime for CuroCell® S.A.C. is estimated to be 1 year.

5.6 Disassembly and recycling

The cushion is completely recyclable and must be sorted as "combustible waste". No disassembly is necessary.

5.7 Returns and complaints

Complaints must be referred to Care of Sweden or the local dealer. Products are to be returned in accordance with our instructions and may only be returned once a complaint has been submitted to Care of Sweden or the local dealer.

5.8 Symbol definitions



Article No



Manufacturer

XXXX-XX-XX

Year-Month-Day



Read the instructions
for use



CE-marked in accordance with
Medical Device Regulation (EU)
2017/745



Recycling



Recommended user weight



Machine wash at 95 °C



Chlorin



Tumble dry



Drip dry



Do not iron



Do not dry clean



Wipe off



UDI



Distributed by



Medical device

6 Accessories

A range of accessories are available. These are listed below.

Levelling plate

If you need to level out the seat cushion, an insert plate is available that can be placed inside the cover against the underside of the cushion. See chapter 2.2.

Covers for Curera® Consil, Curera® Contur, Curera® Contur EX, Curera® Plantur, Curera® Concept, Curera® Concept EX, Curera® Concept Latus, Curera® Hemi, Curera® Tremix, Curera® Contur Children:

Hygienic cover (Consil)	80xxyy-01*
Hygienic cover (Contur/Contur EX)	50xxyy-04*
Hygienic cover (Concept/Concept EX)	54xxyy-02*
Hood Cover (Consil)	80xxyy-00*
Standard cover (Consil)	80xxyy-02*
Standard cover (Contur)	50xxyy-00*
Standard cover (Plantur)	53xxyy-00*
Standard cover (Contur Barn)	50xxyy-01*
Standard cover (Concept)	54xxyy-00*
Standard cover (Concept Latus)	58xxyy-00*
Standard cover (Hemi vänster)	50xxyy-55*
Standard cover (Hemi höger)	50xxyy-50*
Standard cover (Hemi dubbel)	50xxyy-60*
Delux cover (Tremix)	56xxyy-00*
Onyx cover (Contur)	50xxyy-76*
Onyx+ cover (Contur)	50xxyy-78*
Onyx cover (Concept)	54xxyy-03*
Onyx+ cover (Concept)	54xxyy-04*
Onyx cover (Plantur)	53xxyy-03*
Onyx+ cover (Plantur)	53xxyy-04*

*xxyy in the item numbers designate the size of the product. For example, 804045-00 indicates a Hood cover for the Consil cushion measuring 40x40x5 cm (xx= width, yy=depth).

If you need other accessories, or for more information about our products, please contact Care of Sweden or your local dealer.

Covers for Curera® Seat Cushion:

Hygienic cover 35x35x6 cm	521-035035
Hygienic cover 35x41x6 cm	521-035041
Hygienic cover 40x41x6 cm	521-040041
Hygienic cover 40x46x6 cm	521-040046
Hygienic cover 40x50x6 cm	521-040050
Hygienic cover 45x41x6 cm	521-045041
Hygienic cover 45x46x6 cm	521-045046
Hygienic cover 45x50x6 cm	521-045050
Hygienic cover 50x46x6 cm	521-050046
Hygienic cover 50x50x6 cm	521-050050

Covers for CuroCell® S.A.C.:

Hygiene Cover 45x45x5 cm	CC-711-045045
Hygiene Cover 50x50x5 cm	CC-711-050050



Manufactured by

 care of sweden

SUPPORTING LIFE

Contact:

Phone: + 46 771 106 600

Fax: + 46 325 128 40

Email: export@careofsweden.se

Internet: www.careofsweden.com

Address:

Care of Sweden AB
P.O. Box 146
SE-514 23 Tranemo
SWEDEN

Visit address:

Fabriksgatan 5A
SE-514 33 Tranemo
SWEDEN

Cargo address:

Byns väg 4A
SE-514 33 Tranemo
SWEDEN

Distributed by