


User manual

2017-06-07

Helab Seat cushions

Applies to the following products:

Consil
Contur
Plantur
Consept
Consept Latus
Hemi
Tremix
Contur Barn
Standard Cover
Deluxe Cover
Hood Cover
Hygienic Cover
Levelling plate

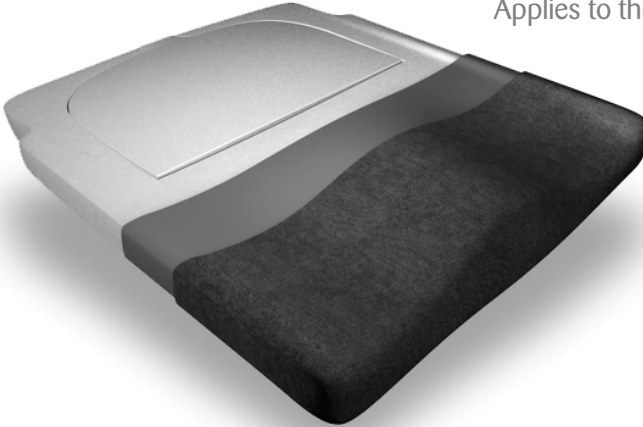

Item no. user guide: 97-001003



7 33 1633 118637



Contents

| | |
|-------------------------------------|----|
| Important safety instructions | 3 |
| Introduction | 3 |
| Product description and performance | 4 |
| Use | 11 |
| Cleaning | 12 |
| Other information | 13 |
| Accessories | 15 |

IMPORTANT SAFETY INSTRUCTIONS

Read all instructions prior to use

WARNING!

1. The product must only be used for its intended purpose as described in this user guide. The product may not be combined, fitted or repaired with other parts, accessories or spare parts other than those described in this user guide or other documentation published by Helab.
2. If the cushion is protected by an incontinence cover, avoid combining it with other fully covering protectors.
3. **DO NOT USE** in the vicinity of or in contact with naked flames/hot surfaces, such as fire, burning cigarettes or hot lamps. Although the product has undergone fire testing, it could become damaged if it comes into contact with naked flames.
4. The product must be placed and used so that it does not become trapped or damaged.
5. If the cushion has been exposed to high temperatures, e.g. if it has been stored in direct sunlight, you must check that it is not too hot to sit on prior to use.
6. Always check that the cushion is of the correct size.
7. Before use, check the cushion to ensure that it is not damaged.
8. Avoid damaging the hygiene cover using sharp objects.
9. Extended periods of sitting increases the risk of DVT (deep-vein thrombosis).
10. Regularly check the skin for redness. If redness occurs, stop using the cushion immediately and contact your prescriber.
11. The cushion is designed to be used in a wheelchair or a chair with back and arm supports. When using in a wheelchair with a suspension kit, a balancing plate is available as an accessory.
12. Cushions must always be used with a cover.
13. Check to ensure that the cover fits the cushion well - it must always be a good fit.

1 Introduction



Always read the user manual prior to use.

1.1 General information

These cushions are medical devices in accordance to MDD 93/42/EEC. According to statutory regulations made by the authorities regarding medical devices, the manufacturer is required to report all accidents or incidents involving these products. We would be very grateful for all information involving accidents or incidents relating to our products, by reporting them immediately to us, here at Care of Sweden.

1.2 Intended use

Helab seat cushions are a range of Swedish-made pressure redistribution cushions that are designed for stability and good sitting comfort. The cushions are intended for users with body sensation who can change their sitting position and are at a low risk of developing pressure ulcers.

The cushion must always be disinfected between users.

1.3 Other information

To achieve the best possible result for this product it is recommended that a well-planned and well-structured working environment be used. In our guideline "Action plan" you will find examples which can be applied and put into practice when using our products.

2 Product description and performance

Helab seat cushions are available in a large numbers of different designs and sizes. The cushion can be used in all segments of the healthcare market and are especially suitable for use with wheelchairs. For details of the standard sizes available. See paragraph 2.2. For other sizes, please contact Care of Sweden or your local dealer.

2.1 Covers

All covers are easy to manage and care for, and conform to stringent requirements in respect of cleaning and hygiene. For more information about cleaning, see paragraph 4.

Standard Cover - A detachable plush polyester cover (100% polyester) with a zipper on the rear edge. An elastic and flexible material on the top side that prevents the user sliding and an underside made from an anti-slip material.

Hygienic cover – A liquid proof cover that is easy to maintain. An elastic and flexible cover with an anti-slip underside - detachable with a zipper on the rear edge.

Hood Cover - Made from 2-way stretch polyester. The fabric is highly flexible and resistant to friction. It is elasticated on all sides, is detachable and easy to remove and fit when washing.

Deluxe Cover - A detachable cover with plush polyester outside and a self-ventilating material inside. Zipper on rear edge. An elastic and flexible material on the top side that prevents the user sliding and an underside made from an anti-slip material.

2.2 Product descriptions

Helab seat cushions are characterised by a high level of comfort and good functionality. They are easy to clean and, with their rounded rear edges and recesses to accommodate wheelchair back supports, they deliver an excellent fit. Markings on the cushions make them easy to position.

Consil

A pressure redistributing and anatomically shaped seat cushion.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Consil, including Hood cover:

| Item no. | Size (WxDxH) | Rec. Max user weight/Kg | Item no. | Size (WxDxH) | Rec. Max user weight/Kg |
|-----------|--------------|-------------------------|-----------|--------------|-------------------------|
| 803535-40 | 35x35x5 cm | 60 | 804245-40 | 42x45x5 cm | 125 |
| 803540-40 | 35x40x5 cm | 60 | 804250-40 | 42x50x5 cm | 125 |
| 803740-40 | 37x40x5 cm | 90 | 804540-40 | 45x40x5 cm | 125 |
| 803745-40 | 37x45x5 cm | 90 | 804545-40 | 45x45x5 cm | 125 |
| 804040-40 | 40x40x5 cm | 90 | 804550-40 | 45x50x5 cm | 125 |
| 804045-40 | 40x45x5 cm | 90 | 804740-40 | 47x40x5 cm | 125 |
| 804050-40 | 40x50x5 cm | 90 | 804745-40 | 47x45x5 cm | 150 |
| 804240-40 | 42x40x5 cm | 125 | 804750-40 | 47x50x5 cm | 150 |
| 804245-40 | 42x45x5 cm | 125 | 805040-40 | 50x40x5 cm | 150 |
| 804250-40 | 42x50x5 cm | 125 | 805045-40 | 50x45x5 cm | 150 |
| 804540-40 | 45x40x5 cm | 125 | 805050-40 | 50x50x5 cm | 150 |

Item information Consil, including Standard cover:

| Item no. | Size (WxDxH) | Rec. Max user weight/Kg | Item no. | Size (WxDxH) | Rec. Max user weight/Kg |
|-----------|--------------|-------------------------|-----------|--------------|-------------------------|
| 513035-11 | 30x35x5 cm | 70 | 514740-11 | 47x40x5 cm | 125 |
| 513535-11 | 35x35x5 cm | 70 | 514745-11 | 47x45x5 cm | 150 |
| 513540-11 | 35x40x5 cm | 70 | 514750-11 | 47x50x5 cm | 150 |
| 513545-11 | 35x45x5 cm | 70 | 515040-11 | 50x40x5 cm | 150 |
| 513740-11 | 37x40x5 cm | 100 | 515045-11 | 50x45x5 cm | 150 |
| 513745-11 | 37x45x5 cm | 100 | 515050-11 | 50x50x5 cm | 150 |
| 513750-11 | 37x50x5 cm | 100 | 515240-11 | 52x40x5 cm | 150 |
| 514040-11 | 40x40x5 cm | 110 | 515245-11 | 52x45x5 cm | 150 |
| 514045-11 | 40x45x5 cm | 110 | 515250-11 | 52x50x5 cm | 150 |
| 514050-11 | 40x50x5 cm | 110 | 515540-11 | 55x40x5 cm | 150 |
| 514240-11 | 42x40x5 cm | 125 | 515545-11 | 55x45x5 cm | 150 |
| 514242-11 | 42x42x5 cm | 125 | 515550-11 | 55x50x5 cm | 150 |
| 514245-11 | 42x45x5 cm | 125 | 515555-11 | 55x55x5 cm | 150 |
| 514250-11 | 42x50x5 cm | 125 | 515740-11 | 57x40x5 cm | 150 |
| 514343-11 | 43x43x5 cm | 125 | 515745-11 | 57x45x5 cm | 150 |
| 514540-11 | 45x40x5 cm | 125 | 515750-11 | 57x50x5 cm | 150 |
| 514545-11 | 45x45x5 cm | 125 | 516045-11 | 60x45x5 cm | 150 |
| 514547-11 | 45x47x5 cm | 125 | 516050-11 | 60x50x5 cm | 150 |
| 514550-11 | 45x50x5 cm | 125 | 516545-11 | 65x45x5 cm | 150 |

Contur

A pressure redistributing seat cushion, constructed from an anatomically moulded polyurethane foam base with viscose elastic foam on the underside.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 5 cm

Warranty: 2 years

Fire: SS-EN 1021-1, SS-EN 1021-2, ISO 7176-16

Item information Contur, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|--------------|-------------------------|-----------|--------------|-------------------------|
| 513035-11 | 30x35x5 cm | 70 | 514740-11 | 47x40x5 cm | 125 |
| 513535-11 | 35x35x5 cm | 70 | 514745-11 | 47x45x5 cm | 150 |
| 513540-11 | 35x40x5 cm | 70 | 514750-11 | 47x50x5 cm | 150 |
| 513545-11 | 35x45x5 cm | 70 | 515040-11 | 50x40x5 cm | 150 |
| 513740-11 | 37x40x5 cm | 100 | 515045-11 | 50x45x5 cm | 150 |
| 513745-11 | 37x45x5 cm | 100 | 515050-11 | 50x50x5 cm | 150 |
| 513750-11 | 37x50x5 cm | 100 | 515240-11 | 52x40x5 cm | 150 |
| 514040-11 | 40x40x5 cm | 125 | 515245-11 | 52x45x5 cm | 150 |
| 514045-11 | 40x45x5 cm | 125 | 515250-11 | 52x50x5 cm | 150 |
| 514050-11 | 40x50x5 cm | 125 | 515540-11 | 55x40x5 cm | 150 |
| 514240-11 | 42x40x5 cm | 125 | 515545-11 | 55x45x5 cm | 150 |
| 514242-11 | 42x42x5 cm | 125 | 515550-11 | 55x50x5 cm | 150 |
| 514245-11 | 42x45x5 cm | 125 | 515555-11 | 55x55x5 cm | 150 |
| 514250-11 | 42x50x5 cm | 125 | 515740-11 | 57x40x5 cm | 150 |
| 514343-11 | 43x43x5 cm | 125 | 515745-11 | 57x45x5 cm | 150 |
| 514540-11 | 45x40x5 cm | 125 | 515750-11 | 57x50x5 cm | 150 |
| 514545-11 | 45x45x5 cm | 125 | 516045-11 | 60x45x5 cm | 150 |
| 514547-11 | 45x47x5 cm | 125 | 516050-11 | 60x50x5 cm | 150 |
| 514550-11 | 45x50x5 cm | 125 | 516545-11 | 65x45x5 cm | 150 |

Contur EX

A pressure redistributing cushion, constructed from an anatomically moulded polyurethane foam base with viscose elastic foam on the underside.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, elastic foam

Height: 5 cm

Warranty: 2 years

Fire: SS-EN 1021-1, SS-EN 1021-2, ISO 7176-16

Item information Contur EX, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|--------------|-------------------------|-----------|--------------|-------------------------|
| 513535-25 | 35x35x5 cm | 70 | 514540-25 | 45x40x5 cm | 125 |
| 513540-25 | 35x40x5 cm | 70 | 514545-25 | 45x45x5 cm | 125 |
| 513740-25 | 37x40x5 cm | 100 | 514550-25 | 45x50x5 cm | 125 |
| 513745-25 | 37x45x5 cm | 100 | 514740-25 | 47x40x5 cm | 125 |
| 514040-25 | 40x40x5 cm | 125 | 514745-25 | 47x45x5 cm | 150 |
| 514045-25 | 40x45x5 cm | 125 | 514750-25 | 47x50x5 cm | 150 |
| 514050-25 | 40x50x5 cm | 125 | 515040-25 | 50x40x5 cm | 150 |
| 514240-25 | 42x40x5 cm | 125 | 515045-25 | 50x45x5 cm | 150 |
| 514245-25 | 42x45x5 cm | 125 | 515050-25 | 50x50x5 cm | 150 |
| 514250-25 | 42x50x5 cm | 125 | | | |

Plantur

A pressure redistributing seat cushion, constructed from anatomically moulded polyurethane foam with a flat top side and a viscose elastic foam insert in the underside.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Plantur, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|-----------------|----------------------------|-----------|-----------------|----------------------------|
| 523035-20 | 30x35x5 cm | 70 | 524250-20 | 42x50x5 cm | 125 |
| 523535-20 | 35x35x5 cm | 70 | 524540-20 | 45x40x5 cm | 125 |
| 523540-20 | 35x40x5 cm | 70 | 524545-20 | 45x45x5 cm | 125 |
| 523545-20 | 35x45x5 cm | 70 | 524550-20 | 45x50x5 cm | 125 |
| 523740-20 | 37x40x5 cm | 100 | 524740-20 | 47x40x5 cm | 150 |
| 523745-20 | 37x45x5 cm | 100 | 524745-20 | 47x45x5 cm | 150 |
| 523750-20 | 37x50x5 cm | 125 | 525040-20 | 50x40x5 cm | 150 |
| 524040-20 | 40x40x5 cm | 125 | 525045-20 | 50x45x5 cm | 150 |
| 524045-20 | 40x45x5 cm | 125 | 525050-20 | 50x50x5 cm | 150 |
| 524050-20 | 40x50x5 cm | 125 | 525540-20 | 55x40x5 cm | 150 |
| 524750-20 | 47x50x5 cm | 150 | 525545-20 | 55x45x5 cm | 150 |
| 524240-20 | 42x40x5 cm | 125 | 525550-20 | 55x50x5 cm | 150 |
| 524245-20 | 42x45x5 cm | 125 | | | |

Consept

A pressure redistributing seat cushion, constructed from an anatomically moulded polyurethane foam base with viscose elastic foam insert on the top side.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Consept, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|-----------------|----------------------------|-----------|-----------------|----------------------------|
| 543535-11 | 35x35x5 cm | 70 | 544545-11 | 45x45x5 cm | 130 |
| 543540-11 | 35x40x5 cm | 70 | 544550-11 | 45x50x5 cm | 130 |
| 543545-11 | 35x45x5 cm | 70 | 544740-11 | 47x40x5 cm | 130 |
| 543740-11 | 37x40x5 cm | 105 | 544745-11 | 47x45x5 cm | 150 |
| 543745-11 | 37x45x5 cm | 105 | 544750-11 | 47x50x5 cm | 150 |
| 544040-11 | 40x40x5 cm | 115 | 545040-11 | 50x40x5 cm | 150 |
| 544045-11 | 40x45x5 cm | 115 | 545045-11 | 50x45x5 cm | 150 |
| 544050-11 | 40x50x5 cm | 115 | 545050-11 | 50x50x5 cm | 150 |
| 544240-11 | 42x40x5 cm | 130 | 545545-11 | 55x45x5 cm | 150 |
| 544245-11 | 42x45x5 cm | 130 | 545550-11 | 55x50x5 cm | 150 |
| 544250-11 | 42x50x5 cm | 130 | 545745-11 | 57x45x5 cm | 150 |
| 544540-11 | 45x40x5 cm | 130 | | | |

Concept EX

A pressure redistributing cushion, constructed from an anatomically moulded polyurethane foam base with viscose elastic foam the topside.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, elastic foam

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Concept EX, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|-----------------|----------------------------|-----------|-----------------|----------------------------|
| 543535-20 | 35x35x5 cm | 70 | 544540-20 | 45x40x5 cm | 130 |
| 543540-20 | 35x40x5 cm | 70 | 544545-20 | 45x45x5 cm | 130 |
| 543740-20 | 37x40x5 cm | 105 | 544550-20 | 45x50x5 cm | 130 |
| 543745-20 | 37x45x5 cm | 105 | 544740-20 | 47x40x5 cm | 130 |
| 544045-20 | 40x45x5 cm | 115 | 544745-20 | 47x45x5 cm | 150 |
| 544050-20 | 40x50x5 cm | 115 | 545040-20 | 50x40x5 cm | 150 |
| 544240-20 | 42x40x5 cm | 130 | 545045-20 | 50x45x5 cm | 150 |
| 544250-20 | 42x50x5 cm | 130 | 545050-20 | 50x50x5 cm | 150 |

Concept Latus

A pressure redistributing seat cushion designed for heavier users. It is constructed from an anatomically moulded polyurethane foam base with a viscose elastic foam insert from the top side.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 7 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Concept Latus, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|-----------------|----------------------------|-----------|-----------------|----------------------------|
| 585045-11 | 50x45x7 cm | 250 | 585750-11 | 57x50x7 cm | 260 |
| 585050-11 | 50x50x7 cm | 250 | 585755-11 | 57x55x7 cm | 260 |
| 585245-11 | 52x45x7 cm | 250 | 586050-11 | 60x50x7 cm | 260 |
| 585250-11 | 52x50x7 cm | 250 | 586055-11 | 60x55x7 cm | 260 |
| 585545-11 | 55x45x7 cm | 250 | 586550-11 | 65x50x7 cm | 270 |
| 585550-11 | 55x50x7 cm | 250 | 586555-11 | 65x55x7 cm | 270 |

Hemi

Hemi is a specially-designed seat cushion with a protected design (Reg.no. 79436) and suitable for people who need to be able to propel themselves in their wheelchair using their legs. Three versions are available - left, right and double.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 5 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Hemi left, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|-----------------|----------------------------|-----------|-----------------|----------------------------|
| 513535-55 | 35x35x5 cm | 70 | 514545-55 | 45x45x5 cm | 130 |
| 513540-55 | 35x40x5 cm | 70 | 514550-55 | 45x50x5 cm | 130 |
| 514040-55 | 40x40x5 cm | 115 | 515040-55 | 50x40x5 cm | 150 |
| 514045-55 | 40x45x5 cm | 115 | 515045-55 | 50x45x5 cm | 150 |
| 514240-55 | 42x40x5 cm | 130 | 515050-55 | 50x50x5 cm | 150 |
| 514245-55 | 42x45x5 cm | 130 | 516050-55 | 60x50x5 cm | 150 |
| 514540-55 | 45x40x5 cm | 130 | | | |

Item information Hemi right, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|-----------------|----------------------------|-----------|-----------------|----------------------------|
| 513535-50 | 35x35x5 cm | 70 | 514545-50 | 45x45x5 cm | 130 |
| 513540-50 | 35x40x5 cm | 70 | 514550-50 | 45x50x5 cm | 130 |
| 514040-50 | 40x40x5 cm | 115 | 515040-50 | 50x40x5 cm | 150 |
| 514045-50 | 40x45x5 cm | 115 | 515045-50 | 50x45x5 cm | 150 |
| 514240-50 | 42x40x5 cm | 130 | 515050-50 | 50x50x5 cm | 150 |
| 514245-50 | 42x45x5 cm | 130 | 516050-50 | 60x50x5 cm | 150 |
| 514540-50 | 45x40x5 cm | 130 | | | |

Item information Hemi double, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|-----------------|----------------------------|-----------|-----------------|----------------------------|
| 513535-60 | 35x35x5 cm | 70 | 514545-60 | 45x45x5 cm | 130 |
| 513540-60 | 35x40x5 cm | 70 | 514550-60 | 45x50x5 cm | 130 |
| 514040-60 | 40x40x5 cm | 115 | 515040-60 | 50x40x5 cm | 150 |
| 514045-60 | 40x45x5 cm | 115 | 515045-60 | 50x45x5 cm | 150 |
| 514240-60 | 42x40x5 cm | 130 | 515050-60 | 50x50x5 cm | 150 |
| 514245-60 | 42x45x5 cm | 130 | 516050-60 | 60x50x5 cm | 150 |
| 514540-60 | 45x40x5 cm | 130 | | | |

Tremix

An anatomically-designed seat cushion constructed in multiple layers and used to prevent pressure ulcers.

Classification: ISO code 04 33 03 (pressure relief)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 7 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Tremix, including Delux cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|--------------|-------------------------|-----------|--------------|-------------------------|
| 563535-11 | 35x35x7 cm | 70 | 564740-11 | 47x40x7 cm | 130 |
| 563540-11 | 35x40x7 cm | 70 | 564745-11 | 47x45x7 cm | 150 |
| 563740-11 | 37x40x7 cm | 105 | 564750-11 | 47x50x7 cm | 150 |
| 563745-11 | 37x45x7 cm | 105 | 565040-11 | 50x40x7 cm | 150 |
| 564040-11 | 40x40x7 cm | 115 | 565045-11 | 50x45x7 cm | 150 |
| 564045-11 | 40x45x7 cm | 115 | 565050-11 | 50x50x7 cm | 150 |
| 564050-11 | 40x50x7 cm | 115 | 564240-11 | 52x40x7 cm | 150 |
| 564240-11 | 42x40x7 cm | 130 | 565245-11 | 52x45x7 cm | 150 |
| 564245-11 | 42x45x7 cm | 130 | 565250-11 | 52x50x7 cm | 150 |
| 564250-11 | 42x50x7 cm | 130 | 565540-11 | 55x40x7 cm | 150 |
| 564540-11 | 45x40x7 cm | 130 | 565545-11 | 55x45x7 cm | 150 |
| 564545-11 | 45x45x7 cm | 130 | 565550-11 | 55x50x7 cm | 150 |
| 564550-11 | 45x50x7 cm | 130 | | | |

Contur Children

A pressure redistributing seat cushion, constructed from an anatomically moulded polyurethane foam base with a viscose elastic foam insert on the top side.

Classification: ISO code 18 10 06 (pressure redistributing)

Material in cushion: Polyurethane foam, viscose elastic foam, vulcanised rubber

Height: 4 cm

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Contur Child, including Standard cover:

| Item no. | Size (WxDxH) | Rec. max user weight/Kg | Item no. | Size (WxDxH) | Rec. max user weight/Kg |
|-----------|--------------|-------------------------|-----------|--------------|-------------------------|
| 512525-12 | 25x25x4 cm | 40 | 513035-12 | 30x35x4 cm | 60 |
| 512530-12 | 25x30x4 cm | 40 | 513335-12 | 33x35x4 cm | 60 |
| 512725-12 | 27x25x4 cm | 60 | 513535-12 | 35x35x4 cm | 60 |
| 512730-12 | 27x30x4 cm | 60 | 513540-12 | 35x40x4 cm | 60 |
| 513030-12 | 30x30x4 cm | 60 | 514040-12 | 40x40x4 cm | 60 |
| 513034-12 | 30x34x4 cm | 60 | | | |

Levelling Plate (Accessories)

There are two types of levelling plate, one placed inside the cover under the cushion, with the rounded rear edge facing backwards and the bevelled underside facing down against the underside of the cover. The other is placed under the wheelchair cushion to fill the gap between the cushion and sling seat, the underside is made from an anti-slip material. It can be used with all cushions except for the Tremix, which already have this feature.

Material in cushion: Cellpolyeten

Warranty: 2 years

Fire: EN 1021-1, EN 1021-2, ISO 7176-16

Item information Levelling plate (without anti-slip):

| Item no. | Size (WxDxH) | Item no. | Size (WxDxH) | Item no. | Size (WxDxH) |
|-----------|--------------|-----------|--------------|-----------|--------------|
| 583535-30 | 35x35x2,5 cm | 584540-30 | 45x40x2,5 cm | 584750-30 | 47x50x2,5 cm |
| 584040-30 | 40x40x2,5 cm | 584545-30 | 45x45x2,5 cm | 585045-30 | 50x45x2,5 cm |
| 584045-30 | 40x45x2,5 cm | 584550-30 | 45x50x2,5 cm | 585050-30 | 50x50x2,5 cm |
| 584240-30 | 42x40x2,5 cm | 584740-30 | 47x40x2,5 cm | 585550-30 | 55x50x2,5 cm |
| 584245-30 | 42x45x2,5 cm | 584745-30 | 47x45x2,5 cm | 585555-30 | 55x55x2,5 cm |

Item Information Levelling Plate (with anti-slip).

| Item no. | Size (WxDxH) | Item no. | Size (WxDxH) |
|-----------|--------------|-----------|--------------|
| 583040-20 | 30x40x2,5 cm | 584545-20 | 45x45x2,5 cm |
| 583540-20 | 35x40x2,5 cm | 584550-20 | 45x50x2,5 cm |
| 584040-20 | 40x40x2,5 cm | 584750-20 | 47x50x2,5 cm |
| 584045-20 | 40x45x2,5 cm | 585045-20 | 50x45x2,5 cm |
| 584540-20 | 45x40x2,5 cm | 585050-20 | 50x50x2,5 cm |
| | | 585550-20 | 55x50x2,5 cm |

Other sizes?

Do you request other sizes please contact Care of Sweden or your local dealer.

3 Use


When unpacking, check to ensure that the item is not damaged. Remove the plastic bag and any protective material. Avoid using a knife as this risks damaging the product.

1. Check that the cushion fits correctly in the cover. Follow the markings on the cover and cushion.
2. Place the cushion as indicated by the markings on the cushion in the wheelchair or armchair. Check that the cushion is the correct size for the chair seat.
3. The user can now start using the cushion.

Take Note! If the cushion Tremix has been stored in a temperature below + 10 °C, it shall be stored at room temperature for at least 1 h before being put into service.


Levelling Plate (without anti-slip)

Place the plate inside the cover, under the cushion, with the rounded rear edge facing backwards and the bevelled underside facing down against the underside of the cover. The plate is also labelled as to how it should be placed.

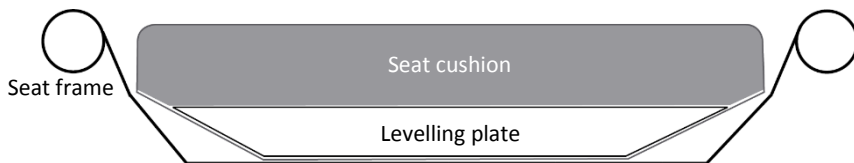

Levelling Plate (with anti-slip)

Place the plate under the wheelchair cushion with the anti-slip material facing downwards.

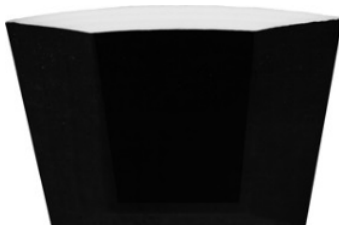

4 Cleaning

It is important to follow the instructions below before using the product for a different user. Consult the hygiene responsible or the supplier for help and instructions when unsure.

NOTE!

Carefully follow local instructions and cleaner usage instructions.

Standard Cover, Deluxe Cover and Hood Cover



The cover must be completely dry before being refitted. Pull the zipper (if the cover has one).

Hygienic Cover



Covers can be wiped off with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70 %, oxidating solutions or chlorine max 1 %.

Excrements and blood spots must be removed immediately with cold water. Carefully follow local instructions and usage instructions for the cleaner.



Chlorine and phenol-based cleaners could have a negative impact on the PU surface and should be avoided. If chlorine is used, we recommend a mixture of max. 1%.

CUSHION - All cushions (except Consil and Levelling plate)



Clean the affect area with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70%, oxidating solutions or chlorine max 1 %.

Let the foam core dry in a warm, ventilated area, without direct exposure of sunlight. The foam core needs to be completely dry before it is used again.

Consil



Clean the affect area with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70 %, oxidating solutions or chlorine max 1 %.

Press out the water carefully. Let the foam core dry in a warm, ventilated area, without direct exposure of sunlight. The foam core needs to be completely dry before it is used again.

Levelling plate (Accessory)



Clean the affect area with a mild cleaning solution for this purpose such as: alcohol with or without tensides, Isopropanol 70%, oxidating solutions or chlorine max 1 %.

It needs to be completely dry before it is used again.

5 Other information

5.1 Storage

Store the cushion flat and with its cover on. The cushion should be stored in a dry, room temperature area. Store the cushion so that it cannot be exposed to shocks, impacts or other mechanical influences. Avoid exposing the cushion to direct sunlight.

5.2 Labelling

The cushion is labelled with date of manufacture, order number, item number, size, care and maintenance advice, EAN number. It is also marked showing how it must be positioned in the chair.

5.3 Warranty

The warranty is specified under paragraph 2.2.

The warranty does not apply to normal wear and tear or product impairment or damage stemming from negligence or improper handling/care.

5.4 Service and maintenance

The cushion requires no service or maintenance other than cleaning as described in this user manual.

5.5 Life expectance

This product's expected life is estimated to be 2 years.

5.6 Disassembly and recycling

The cushion is completely recyclable and must be sorted as "combustible waste". No disassembly is necessary.

5.7 Returns and complaints

Complaints must be referred to Care of Sweden or the local dealer. Products are to be returned in accordance with our instructions and may only be returned once a complaint has been submitted to Care of Sweden or the local dealer.

5.8 Symbol definitions



Article No



Manufacturer

XXXX-XX-XX

Year-Month-Day



Read the user manual



CE marked acc. to
MDD 93/42/EEC



Recycling



Recommended user weight



Machine wash at 95 °C



Chlorine



Tumble dry



Drip dry



Do not iron



Do not dry clean



Wipe off

6 Accessories

A range of accessories is available for these cushions. These are listed below.

Levelling plate

If you need to level out the seat cushion, an insert plate is available that can be placed inside the cover against the underside of the cushion. See page 11 for more information.

Cover

| | |
|-------------------------------------|------------|
| Hygienic cover (Consil) | 80xxyy-01* |
| Hygienic cover (Contur/Contur EX) | 50xxyy-04* |
| Hygienic cover (Consept/Consept EX) | 54xxyy-02* |
| Hood Cover (Consil) | 80xxyy-00* |
| Standard cover (Consil) | 80xxyy-02* |
| Standard cover (Contur) | 50xxyy-00* |
| Standard cover (Plantur) | 53xxyy-00* |
| Standard cover (Contur Barn) | 50xxyy-01* |
| Standard cover (Consept) | 54xxyy-00* |
| Standard cover (Consept Latus) | 58xxyy-00* |
| Standard cover (Hemi vänster) | 50xxyy-55* |
| Standard cover (Hemi höger) | 50xxyy-50* |
| Standard cover (Hemi dubbel) | 50xxyy-60* |
| Delux cover (Tremix) | 56xxyy-00* |

*xxyy in the item numbers designate the size of the product. For example, 804045-00 indicates a Hood cover for the Consil cushion measuring 40x40x5 cm (xx= width, yy=depth).

If you need other accessories, or for more information about our products, please contact Care of Sweden or your local dealer.



Manufactured by

care of sweden
Supporting Life

Contact:

Phone: + 46 771 106600
Fax: + 46 325 12840
Email: info@careofsweden.se
Internet: www.careofsweden.com

Address:

Care of Sweden AB
P.O. Box 146
SE-514 23 Tranemo
SWEDEN

Visit address:

Fabriksgatan 5A
SE-514 33 Tranemo
SWEDEN

Cargo address:

Byns väg 4A
SE-514 33 Tranemo
SWEDEN

Supplied by