

Optimal Pillow Case Cover & Hygiene Cover

Item number user manual: 95-001668



7 33 1345 079738



IMPORTANT SAFETY INSTRUCTIONS

Read all instructions prior to use

WARNING!

1. The product must only be used for its intended purpose as described in this manual. The product may not be combined, fitted or repaired with other parts, accessories or spare parts other than those described in this manual or other documentation published by Care of Sweden.
2. Do not use in the vicinity of or in contact with fire sources/hot surfaces, such as fire, burning cigarettes or hot lamps. Although the product has undergone fire testing, it could become damaged if it comes into contact with naked flames.
3. Position and use the product in a manner that prevents it from being crushed or damaged. Pay particular attention where bed rails are used. Avoid injury caused by sharp objects that may affect the hygiene cover's function.
4. The hygiene cover is water and airtight, but vapour permeable. Make sure that the patient is positioned correctly to avoid the risk of suffocation.
5. Avoid combining the product with other full-cover plastic protectors. The cover's vapour permeability may be affected.



Always read the user manual prior to use.

General information

This product is a medical device and is CE-marked in accordance with MDD 93/42/EEC. Pursuant to the Swedish Medical Products Agency's regulations on medical devices, the manufacturer is obliged to report accidents and near-accidents. We would therefore be grateful if any accidents and near-accidents involving our products could be reported immediately to Care of Sweden.

Product description Pillow case cover

Removable pillow case covers made of polyurethane-coated polyester fabric. An elastic, flexible fabric that is waterproof yet vapour permeable. Complies with the applicable parts of the following standards: SS 876 00 01, SS 876 00 02, SS 876 00 04, SS-EN ISO 12182, SS-EN ISO 14971, SS-EN ISO 10993-1, SS-EN ISO 12952, EN 14126.

Intended use Pillow case cover

Removable hygiene cover with very good hygiene standard. Available in sizes 35x45, 50x60, 50x70 cm.

Product description Hygiene cover

Optimal Hygiene Cover is a simple, durable and functional mattress protector with a good fit and flexibility. The hygiene cover is available in several different sizes. Suitable for interior sprung mattresses, standard mattresses and mattress overlays. Easy to take travelling. Made of polyurethane-coated polyester fabric, a material that is soft and elastic. The hygiene cover is completely watertight yet vapour permeable, reducing the risk of skin maceration. Complies

with the applicable parts of the following standards: SS 876 00 01, SS 876 00 02, SS 876 00 04, SS-EN ISO 12182, SS-EN ISO 14971, SS-EN ISO 10993-1, SS-EN ISO 12952, EN 14126.

Action plan for pressure ulcers

For the best possible results when using these products, we recommend a structured and planned approach. Examples that can be applied to the use of our products can be found in the “Action plan for pressure ulcers” guidelines. This information can be downloaded from our website www.careofsweden.com or ordered via customer service. See contact information on the last page.

Cleaning and reconditioning

It is important to follow the instructions below before the product is used for another user.

Consult your hygiene manager or Care of Sweden for help and instructions if you are unsure.

Check the hygiene cover each time the product is cleaned. If damaged, it must be replaced or repaired.

Instructions



Covers can be wiped off with a mild cleaning solution for this purpose: such as alcohol with or without tensides, Isopropanol 70 %, oxidating solutions or chlorine max 1 %.

Excrements and blood spots must be removed immediately with cold water. Carefully follow local instructions and usage instructions for the cleaner.



The cover can be machine-washed at max. 95°C, and tumble-dried or drip-dried.



Chlorine and phenol-based cleaners may adversely affect the PU surface and should be avoided. If chlorine is used, we recommend a maximum concentration of 1%.

Storage

The product should be stored in a dry area at room temperature.

Labelling

Labelled with its product name, date of manufacture, item number, size, care and maintenance advice, EAN number and space for your own labels (such as ward number and name).

Options, Pillow case cover

Welded seams

Plastic zip

Service and maintenance

The product requires no service or maintenance other than cleaning as described in this user manual.

Service life

The estimated service life of this product is two years.

Repair

For instructions, please contact Care of Sweden, telephone +46 (0)771-106 600.

Disassembly and recycling

The product is completely recyclable and must be sorted as “combustible waste”. No disassembly is necessary.

Returns and complaints

Complaints should be submitted to Care of Sweden. Products are to be returned in accordance with our instructions and may only be returned once a complaint has been submitted to Care of Sweden. Transportdamage must be reported to the transport company.

Key to symbols



Item number



Manufacturer

XXXX-XX-XX

Year-Month-Day



Read the user manual



CE-marked in accordance with MDD 93/42/EEC



Recycling



Machine wash at 95°C



Chlorine



Tumble dry



Drip dry



Do not iron



Do not dry clean



Wipe clean



Medical Device



Manufactured by

Contact:

Telephone: +46 (0)771-106 600

Fax: +46 (0)325-128 40

E-mail: info@careofsweden.se

Internet: www.careofsweden.com

Postal address:

Care of Sweden AB

PO Box 146

SE-514 23 Tranemo

Visiting address:

Fabriksgatan 5A

SE-514 33 Tranemo

Deliveries address:

Byns väg 4A

SE-514 33 Tranemo

Urgent matters, outside office hours (weekdays 4.30 pm – 10 pm and weekends 8 am – 5 pm):

Telephone +46 (0)325-101 11